

Let's compare U-drain with Drain Lines & Drain Bags:

Time

To disconnect, drain & discard
Drain Bag - 5 minutes

To disconnect, drain & discard
U-drain - 1 minute

Cost

1 year supply of **Drain Bags £1,400**
1 Year supply of **U-drain £540**
(circa 60% saving)


Safety & Ease of Use

Drain Bags are heavy, cumbersome & a safety concern; could present a chance of back injury or hernia which would result in hospitalisation.

U-drain is light and simple to use.

Environment

U-drain offers a 70% reduction in carbon emissions compared to Drain Bags. Landfill is reduced by 80%.

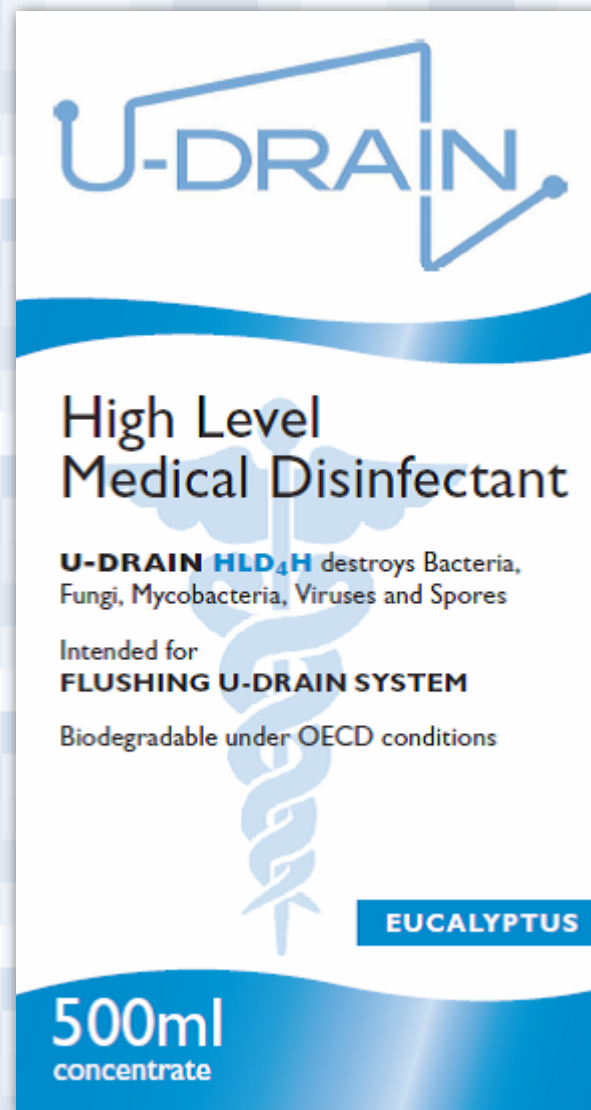

U-drain Limited:
6 Barry Rise, Bowdon,
Altrincham, Cheshire, WA14 3JS

Tel: 0161 941 1969
Email: info@u-drain.co.uk
Web: www.u-drain.co.uk



Automated Peritoneal Dialysis Drainage

Compatible with all Home APD Machines including:
Baxter & Fresenius



We now have an option, no more carrying heavy bags of waste solution to the toilet, sink or shower.

www.u-drain.co.uk

Automated Peritoneal Dialysis Drainage

Basics

U-drain APD is a temporary fitted drainage system. The system is removed once dialysis is stopped

U-drain trained installers will fit the system and remove it when required.

The Wall Socket is positioned next to the APD machine and connected to the House drainage pipe via a discreet 13mm tube.

The Wall Socket contains a one-way valve which stops backflow & odour.

Avoidance of backflow is assured; the one-way valve will withstand high back pressure. The APD machine will not allow solution to backflow.

U-drain Disinfectant is proven to kill Bacteria, Fungi, Mycobacteria, Viruses and Spores.

Emergency procedure

In the very rare event of the system blocking or freezing, or any other form of system failure, please revert to the night drainage bags supplied. Followed by a telephone call to U-drain:

0161 941 1969

At Night

Once the APD set up is complete:

Simply connect the APD Drain Line to the Wall Socket using the U-drain Adaptor as seen in the photo.



In the Morning

Disconnect the U-drain Adaptor from the wall socket and discard as usual.

Fill 100ml syringe with warm water and 20ml of U-drain disinfectant.

Flush solution through the system.

